

# Simply Tranquil™ II



The only all-soft, mini ITC tinnitus noise generator for extended wear comfort.

The Simply Tranquil II is a readywear tinnitus noise generator device utilizing digital signal quality. Patients will appreciate same day fitting convenience - no need for ear impressions!

## Device Data

Both noise generator circuits provide the user with a 0 dB SL noise floor setting.

### GENERAL WHITE NOISE RESPONSE

Source.....OFF  
 Peak.....72.9 dB SPL  
 Peak Frequency.....300 Hz  
 RMS OUT.....80.4 dB SPL  
 Battery Drain.....0.40 mA  
 Battery Life (10A).....200 Hrs.

### GENERAL PINK NOISE RESPONSE

Source.....0.0 dB  
 Peak.....79.7 dB SPL  
 Peak Frequency.....300 Hz  
 RMS OUT.....82.6 dB SPL  
 Battery Drain.....0.40 mA  
 Battery Life (10A).....200 Hrs.

### Device Features

- General white & pink noise
- Multi-function switch
- Tranquil circuitry
- 0 dB SL noise floor
- Low battery warning
- #10 battery
- 200 hours battery life
- Open-ear venting
- Hypoallergenic soft body



Simply Tranquil II




**Offer tinnitus patients general white and pink noise in a soft, extended wear Tranquil™ brand noise generator.**

The Simply Tranquil™ II device is a soft, comfortable noise generator that can be worn all day and throughout the night. It is the smallest, most comfortable low-level noise generator on the market today. The soft, flexible design offers natural sound quality due to the open canal format in a small ITC (In-The-Canal) design.

The memory/volume selector toggle is simple to use and combines two functions into one control. Holding the toggle switch in either direction will result in a smooth memory change; momentary “clicking” the toggle in an upward or downward motion results in discreet 1dB increments in volume.

The Simply Tranquil™ II is powered by a zinc-air #10 battery that provides about 200 hours battery life. It features 0 dB SL noise floor volume control - no noise produced when volume is set at lowest setting. The soft body is medical grade hypoallergenic silicone and will not shrink or harden over time.

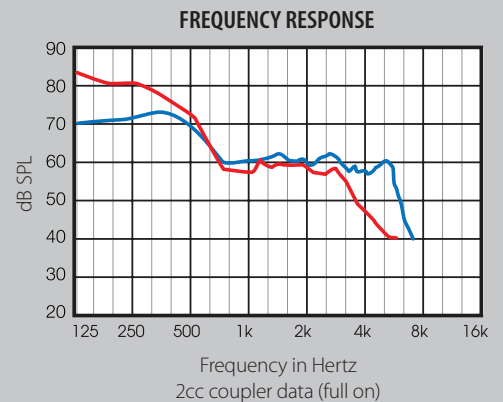

**BLUE CURVE** = General white noise  
**RED CURVE** = General pink noise



175 Brookhollow Esplanade  
 Harahan, LA 70123  
 800.824.3021 504.733.3767

Hearing Made Simple  
[www.generalhearing.com](http://www.generalhearing.com)

